

SPECIMEN COLLECTION AND HANDLING




IMPORTANT!

- Refer to the Package Insert for Warnings and Precautions, Specimen Collection Procedures, Storage and Handling Conditions, and Quality Control Recommendations.


CLIA complexity: MODERATE




- 1 Check the expiration date. Do not use the kit contents beyond the expiry date.

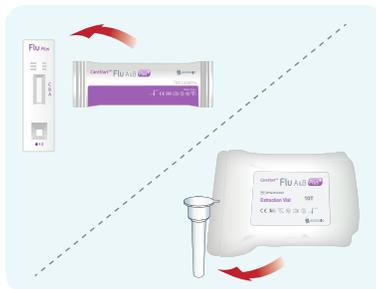


- 2 Remove one test device and one extraction vial from foil pouch.

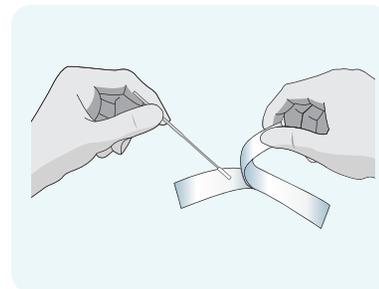


- 3 Remove a nasopharyngeal swab from the pouch.

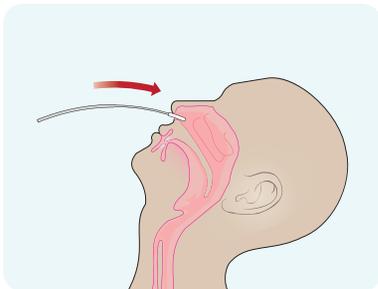

- 4 Place the swab into one of patient's nostrils until it reaches the posterior nasopharynx.

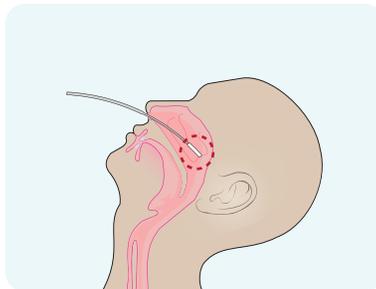

- 5 Slowly rotate 3-5 times the swab over the surface of the posterior nasopharynx.

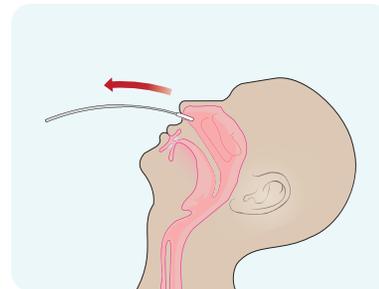

- 6 Remove the swab from the nostril.


TEST PROCEDURES




7 Peel off aluminum foil seal.



8 Place the swab into the extraction vial. Rotate the swab vigorously at least 5 times.

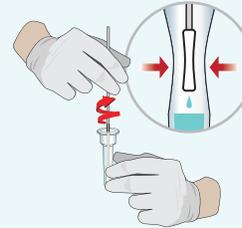

9 Remove the swab with rotating against the extraction vial while squeezing the sides of the vial to release the liquid from the swab. Properly discard the swab.

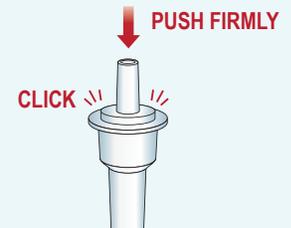

10 Close the vial by pushing the cap firmly onto the vial.



11 Mix thoroughly by flicking the bottom of the tube.



12 Invert the extraction vial and hold the sample vertically above the sample well. Squeeze the vial gently. Allow three (3) drops of sample to fall into the sample well.

Start the timer




Read the result after 10 minutes. The test result should not be read after 15 minutes.




Result Interpretation


Positive Flu A

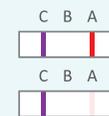

Positive Flu B

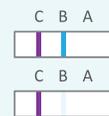

Positive Flu A, B

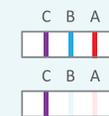

Negative



NOTES

- The color intensity in the test region will vary depending on the amount of influenza virus type A and/or B antigen present in the sample.
- Any faint colored line(s) in the test region(s) should be considered as positive.

Invalid Result

Re-run the test one time using the remaining specimen in the extraction vial.



Access Bio, Inc.

65 Clyde Road, Suite A
Somerset, NJ 08873, USA

Tel: 732-873-4040

Fax: 732-873-4043

Email: info@accessbio.net

Website: www.accessbio.net

Technical Support

Tel: 888-898-1270 (Toll Free)

Email: TShelp@accessbio.net